




Dates and images

Intrauma S.r.l. reserves the right to modify the design and finishing of the products shown and described in the present catalogue without any notice. The images are informative and only regarded as examples. The information provided is purely indicative of the products displayed. Such information does not constitute an accurate description of the characteristics on the part of the manufacturer. We therefore invite our clients to always contact Intrauma S.r.l. to obtain full details on our product's specifications.

© Intrauma - Patented products - July 2015

Specific implants

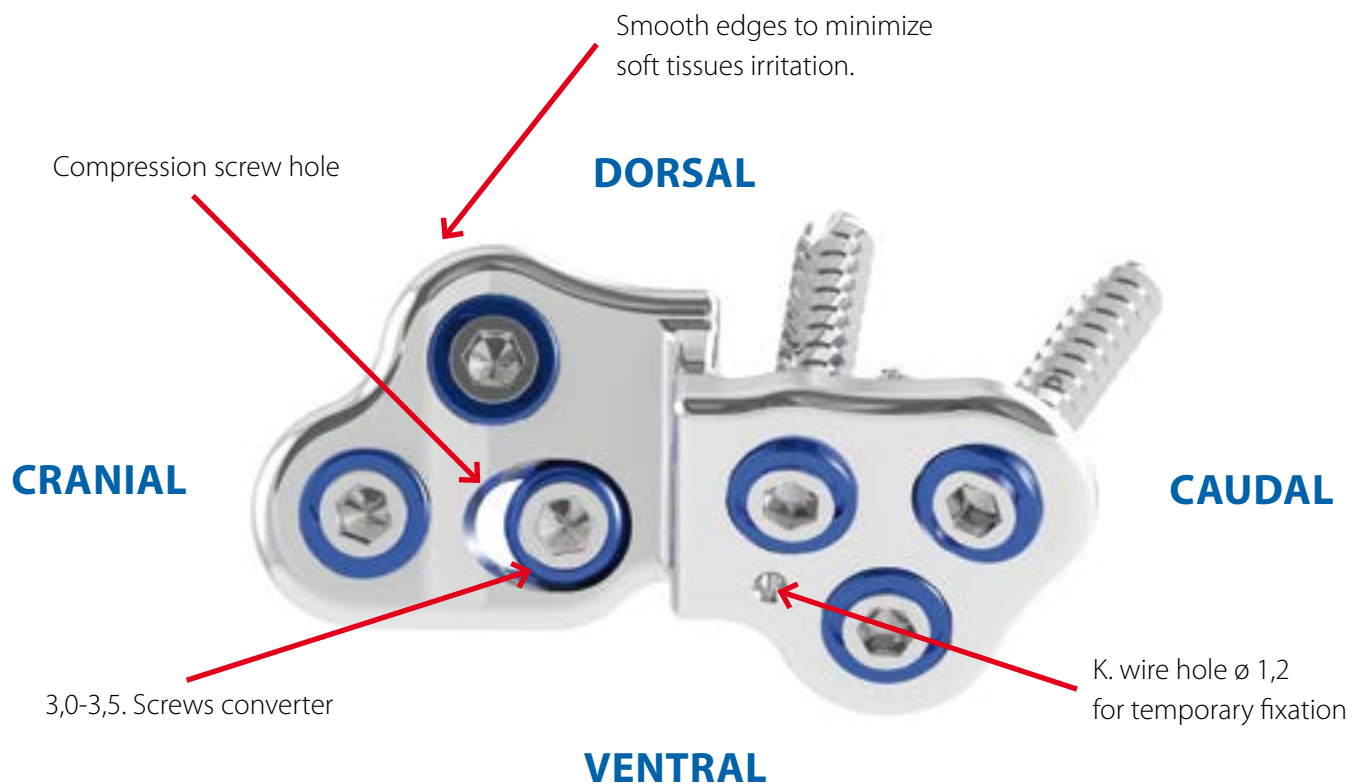


WARNING: The kind of bushing is not compatible with 3,0-3,5 Standard bushing extractor.

DPO Support - Length 40 mm - Thickness 3,0 mm - 30°

Labrador, 5 months, 19,4 kg.

Courtesy of Dr. Aldo Vezzoni



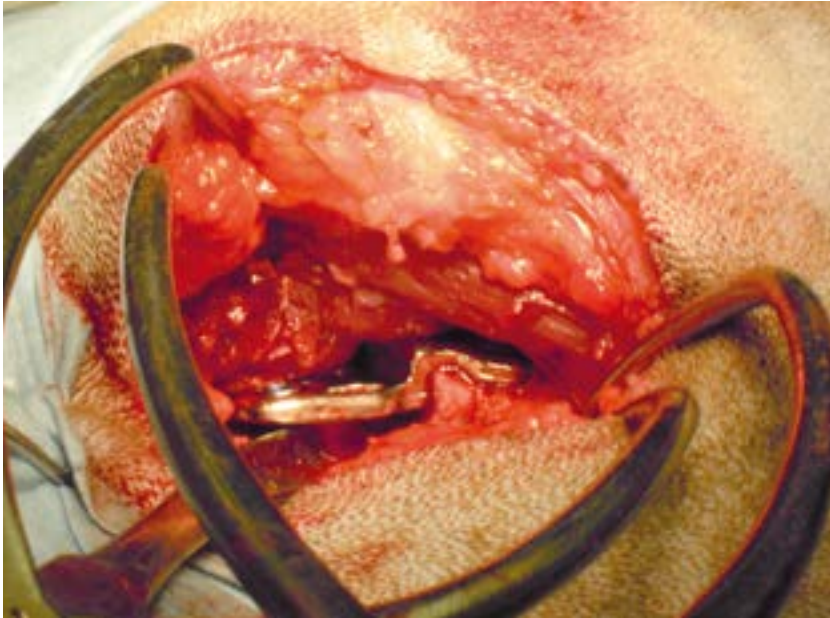


Small implant

Length 40 mm

Thickness 3,0 mm


Fits on more patients even medium size or very young dogs.

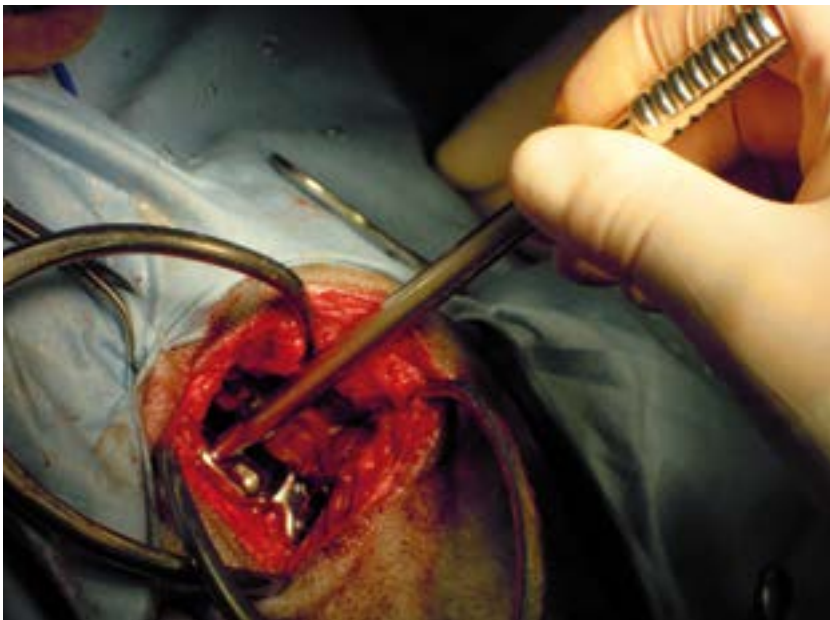

VS383

DPO Positioner

Insert the DPO positioner into the most cranial bushing on the plate. Use the DPO positioner as a lever to rotate implant and bone starting the desired correction; correction will be completed and locked in position through the insertion of a compression screw in the most ventral hole on the plate

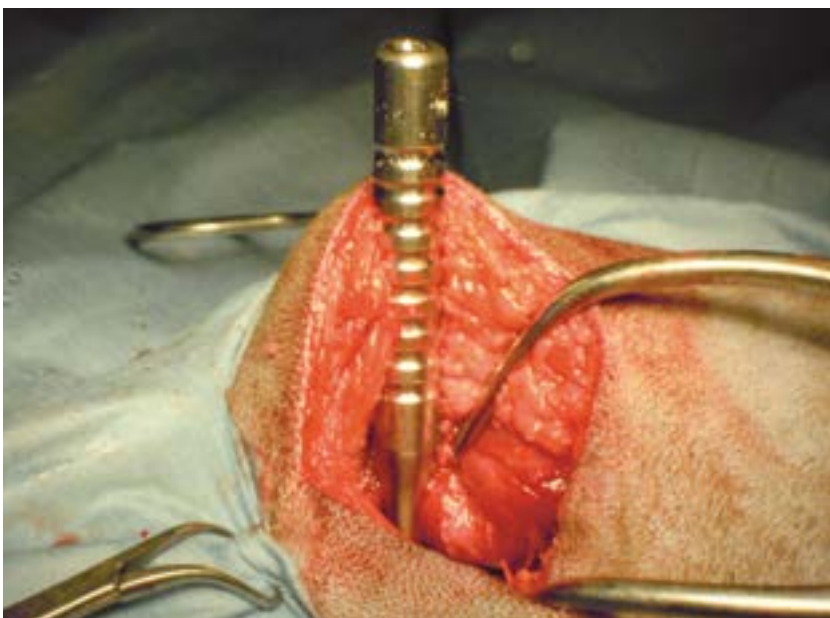

VS382

Long Drill guide for \varnothing 3,5 Screws

Length 85 mm


The drill guide has been designed in cooperation with the University of Turin (prof. Bruno Peirone). It allows to retract soft tissues and facilitates implant positioning.



VS136


3,0 - 3,5 Screws head converter

Insert the FIXIN screw into the conical recess of the converter, pull in order to achieve a cthe coupling.

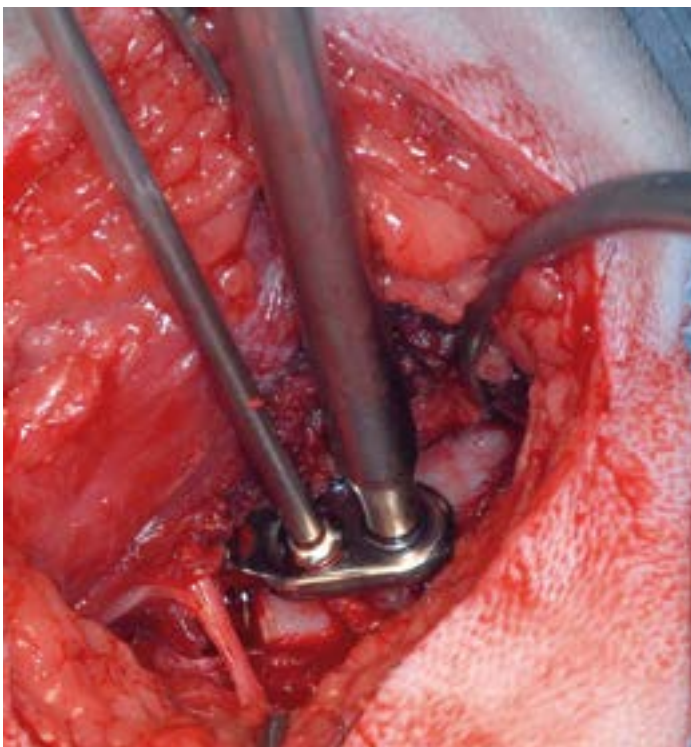

Insert the FIXIN screw, previously assembled to the converte and screw in mantaining the torsion created through the DPO positioner until the rotation is complete and the palte adheres.

Note that any 3,5 mm traditional screw is compatible.

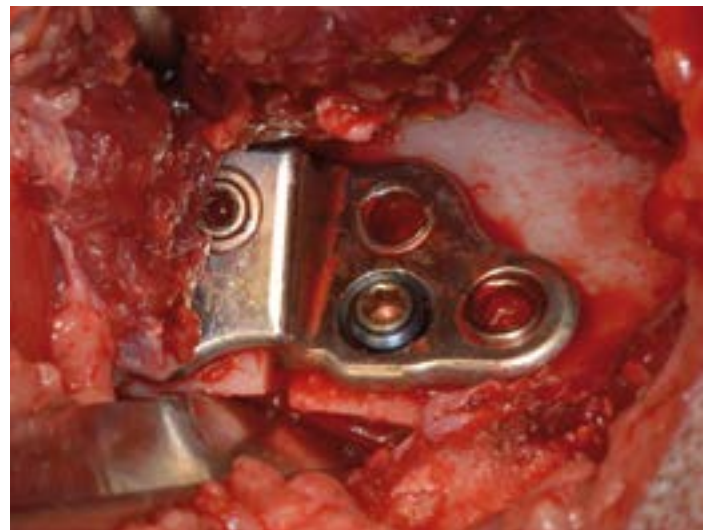

VS355

Reduced Bushing extractor

The Reduced Bushing extractor is specific for low profile bushings.

Ordering information



Autolocking Screws ø 3,5 mm

Ref.	L. mm
V3516	16
V3518	18
V3520	20
V3522	22
V3524	24



Reduced head Cortical Screws ø 3,5 mm

Ref.	L. mm
V3616	16
V3618	18
V3620	20
V3622	22
V3624	24



3,0 - 3,5 Series Instrumentation

Ref.	Description
VS33	Depth meter ø 2,5 - 2,9 mm
VS340	Hexagonal screwdriver ø 2,5 mm
VS35	Bushing carrier/extractor
VS380	Drill guide ø 3,5 mm
VS381	Drill guide ø 3,0 mm
VS390	Drill bit ø 2,9 mm
VS391	Drill bit ø 2,5 mm
VS36	Pinstopper
VSVS	Temporary stab. screw ø 3,5 mm
VS310	Screw rack
VS136	Screws head converter (Included with plates)

3,0 - 3,5 DPO Dedicated Instrumentation

Ref.	Description
VS355	Low profile bushing extractor
VS383	DPO positioner
VS382	Extra long drill guide ø 3,5 mm

3,0-3,5 Series TPO/DPO Supports - Thickness 3,0 mm

Ref.	Description	Length
V3107	TPO/DPO Support 3+3 Holes 20° (L)	40 mm
V3108	TPO/DPO Support 3+3 Holes 20° (R)	40 mm
V3109	DPO Support 3+3 Holes 25° (L)	40 mm
V3110	DPO Support 3+3 Holes 25° (R)	40 mm
V3111	DPO Support 3+3 Holes 30° (L)	40 mm
V3112	DPO Support 3+3 Holes 30° (R)	40 mm
V3113	TPO/DPO Support 3+4 Holes 20° (L)	43 mm
V3114	TPO/DPO Support 3+4 Holes 20° (R)	43 mm
V3115	DPO Support 3+4 Holes 25° (L)	43 mm
V3116	DPO Support 3+4 Holes 25° (R)	43 mm
V3117	DPO Support 3+4 Holes 30° (L)	43 mm
V3118	DPO Support 3+4 Holes 30° (R)	43 mm



*All you need.
With us.*



Intrauma S.r.l. - Via Genova, 19 - 10098 Rivoli (TO) Italia
Tel. +39.011.953.94.96 - Fax +39.011.957.42.91 - fixin@intrauma.com - www.intrauma.com

USA Office: **Starvet LLC** - Suite 317A - 50 Harrison str. - 07030 Hoboken - NJ
P. 201.420.0110 - F. 201.420.0228 - starvet@verizon.net